



290-010 Denniston Dilator Sterilization & Disinfection

Suggested Disinfecting Procedure for the Physician

1. **Cleaning:** Prepare enough enzymatic solution to fully submerge the device and follow the solution cleaning instructions. After cleaning, rinse off with water thoroughly
2. **A. Disinfecting:** use disinfecting solution like CIDEX™ OPA for 12 minutes and rinse with water thoroughly (be sure to follow manufacturer's instructions) **Or**
B. Sterilizing: Recommended Sterilization Parameters

Sterilizer	Exposure Temperature	Exposure Time	Minimum Dry Time
Pre-vacuum	132°C (270°F)	4 Min	20 Min
Gravity Steam	121°C (250°F)	30 Min	30 Min

PRECAUTION: Sterilize/Disinfection before use. Do not insert to a depth of more than 4cm.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

Made in Taiwan



661-296-7200



info@haydenmedical.com

www.haydenmedical.com



19425 Soledad Cyn Rd #411

Santa Clarita, CA 91351